
Dentosept® Clean

Gebrauchsanweisung
Instructions for Use
Mode d'emploi
Instrucciones de uso
Istruzioni d'uso
Gebruikershandleiding

ALPRO
ALPRO MEDICAL GMBH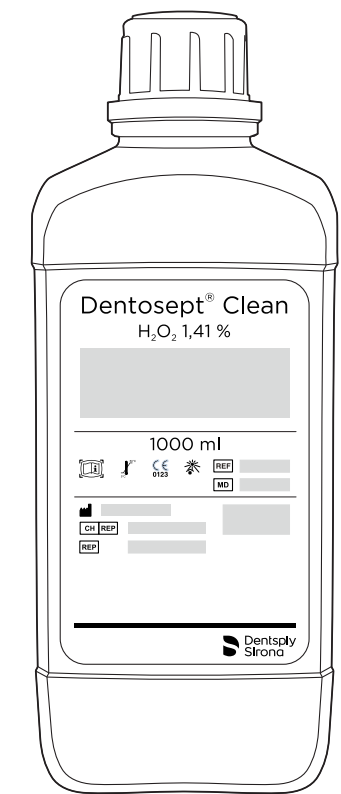

ALPRO MEDICAL GMBH • Germany
78112 St. Georgen • Mooswiesenstr. 9
www.alpro-medical.com • T + 49 7725 9392-0

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EN Instructions for Use

Intended use: Dentosept® Clean is used to maintain the quality of the treatment water and for the disinfection of the treatment water paths in Dentsply Sirona dental treatment centers.

Indication: Reduction of microbial growth and germ reduction in treatment water through continuous inoculation with Dentosept Clean and disinfection of the treatment water lines with undiluted Dentosept Clean for at least 24 hours

Composition: 1.41% hydrogen peroxide, less than 0.1% stabilizers, approx. 98.5% H₂O.

Storage and handling: Application: 1%; Please note the instructions for use for Dentsply Sirona dental treatment centers.

For use only by trained medical personnel
Recommended storage temperature: 5 °C - 50 °C.
Keep away from direct sunlight!

Effectiveness: Bactericidal (EN 13623, Test bacteria: *Legionella pneumophila*) undiluted in 16 hours Preservation (EuAB 5.1.3. Test bacteria: *Pseudomonas aeruginosa*, *Candida albicans*) 1% in 60 minutes

Material compatibility: No incompatibility known. Material compatibility tested on Dentsply Sirona parts.

Information on residual risks: The mixture is not classified as hazardous in the sense of Regulation (EC) No. 1272/2008 [CLP].

Dentosept Clean may not be mixed with other cleaning and disinfecting agents.

Serious adverse events associated with the product must be reported to the manufacturer and the competent authority of the Member State.

Suitable for children from 2.5 years.

Safety recommendation: Avoid contact with skin and eyes. Keep away from children. Do not ingest. Safety data sheet available on request.

Disposal information: Do not reuse the product. Remaining product is to be disposed of as non-hazardous waste according to Waste Directive 2008/98/EC and the national and regional regulations. Do not mix with other waste. Contaminated packaging should optimally be emptied and can then be taken for recycling.

| | |
|-----------|-----------------------|
| DA | Brugsanvisning |
|-----------|-----------------------|

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